better **days** by
atoform[®]

D

GB

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**Badewannenbrett
Bonaire**

CE

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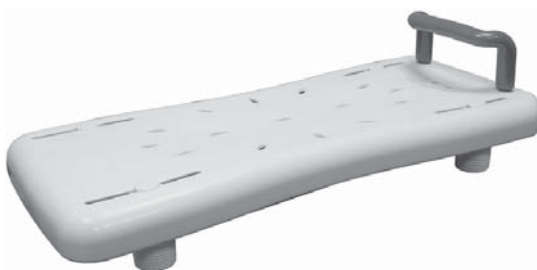
Bath board
587

CE

The following manual provides descriptions relating to the assembly, operation and care of the bath board.

Please consult your doctor prior to using the product if this represents the appropriate aid for you.

If you have any other questions about this bath board, please contact your supplier.

Contents

1.0	Safety instructions	4
1.1	Symbols used	4
1.2	Use in accordance with the requirements	4
1.3	Safety - General notes	4
1.4	Notes on the assembly of the bath boards	5
2.0	Extent of delivery /component name	6
3.0	Assembly	6
3.1	Preparation	6
3.2	Screw down handle	7
3.3	Attach adjustable feet	7
4.0	Attaching the bath board to the bath rim	8
5.0	Cleaning and disinfection	9
5.1	Cleaning	9
5.2	Disinfection	9
6.0	Maintenance / Spareparts	9
6.1	Maintenance	9
6.2	Spare parts	9
7.0	Reuse	10
8.0	Disposal	10
9.0	Name plate	10
10.0	Specifications	10
11.0	Warranty	10

1.0 Safety instructions

1.1 Symbols used

The following symbols are used in the manual:

Caution!

This symbol identifies safety information which notifies you of hazards when dealing with the product

NOTE

You will also find information about dealing with the product under this symbol.

1.2 Use in accordance with the requirements

The **Days Healthcare** bath board offers individuals a seat option in the direction of the bath's longitudinal axis in order to perform daily personal hygiene in the bath. It is intended for individuals whose upper body control and capacity of the upper extremities is still sufficient.

Possible conditions or results of injury are e.g.:

- Considerably reduced function / lack of function of the lower extremities (e.g. amputation, paralysis, joint disease) while upper body control and capacity of the upper extremities is still sufficient.

The bath board is only intended for use in connection with standard bath tubs.

The secure fastening of the bath tub, for which the use of the bath board is intended, is required.

The bath board must only be used in bath tubs in which a secure position of the board is guaranteed.

The bath board must only be used when fully assembled.

1.3 Safety - General notes

Please read the assembly instructions for the bath boards carefully before starting the assembly!

Do not continue to use the bath board if defects are detected during use.

Please take care that:

- the instructions are read by all individuals who will use the bath board.
- all individuals using the bath board have access to the instructions at any time.

Damages resulting from a failure of observing the instructions are excluded from any warranty.

Restrictions of use!

- *Patients weighing more than **190 kg** must not use the bath boards.*

Accident risk!

- *Only use the bath boards for their intended use. They are not replacements for ladders and are not exercise equipment or toys.*
- *The bath boards are not intended for use during examinations or treatments.*
- *Use the bath boards only in technically perfect condition.*
- *Use only spare parts and accessories released by **Days Healthcare**.*
- *Do not carry out structural changes which are not approved by the manufacturer.*
- *The bath boards must not be used in environments in which flammable gases or vapours (e.g. from anaesthetics) can be expected.*
- *Make sure before use that the bath board rests securely on the rim of the bath tub and is secured from slipping.*

1.4 Notes on the assembly of the bath boards

Risk of injury due to wrong assembly!

- *Check the perfect position and firm support of all parts after the assembly.*

Risk of injury!

- *Use only tools appropriate for the work.*
- *Carry out the work only if you are familiar with using the tools employed.*

2.0 Extent of delivery / component name

The bath board is supplied in a box.
Use the following list to check the contents of the package before starting the assembly.
Please contact your medical supply shop if defects are apparent or if parts are missing.

Contents:

- 1 x (1) Bath board
- 1 x (2) Handle
- 2 x (3) Fastening screws with slot
- 4 x (4) Adjustable feet
- 4 x (5) Rubber washer
- 4 x (6) Fastening screws for adjustable feet

3.0 Assembly

Observe the safety notes in chapter 1.3

3.1 Preparation

- ➔ Remove fixing bolts (3 + 4) from handle and feet.

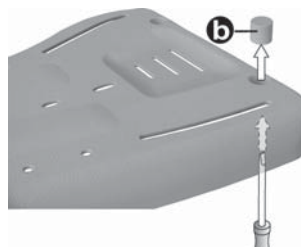
3.2 Screw down handle

Required tools:

Screw driver

- Remove locking caps (b) from handle mounting.

Insert handle


- Insert handle (2) into the holes of the bath board (1).

NOTE

Take care that the handle (a) points to the seat.

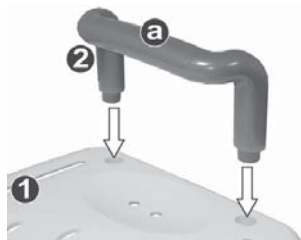Insert handle

- Screw fastening screws (3) into the threaded insert of the handle from below and tighten with screw driver.

NOTE

Tightening too much results in the destruction of the threaded inserts.

Tighten fastening screws

3.3 Attach adjustable feet

- Insert fastening screw (5) from the top into the slot of the bath board.

Insert fastening screw

- ➔ Place rubber washer (5) on fixing bolt from below.
- ➔ Screw adjustable foot (4) onto the fastening screw from below and tighten hand-tight.

- ➔ Assemble all other adjustable feet (4) as described above.

NOTE

Tightening too much results in the destruction of the threaded inserts.

Screw down adjustable foot

4.0 Attaching the bath board to the bath rim

Use the bath board only with bath tubs for which a secure position on the tub rim is guaranteed.

- ➔ Place bath board on the bath tub rim.
- ➔ Align the centre of the bath board.
- ➔ Loosen all adjustable feet (4), push to the tub rim and tighten.

NOTE

The adjustable feet prevent slipping of the bath board to the side.

Attach and secure bath board

Accident risk!

- Make sure before use that the bath board rests securely on the bath tub rim and is secured against slipping.

5.0 Cleaning and disinfection

NOTES

- Do not use tools with sharp edges (knives, metal scrapers) or aggressive solvents for cleaning.
- Observe the instructions on the cleaning agents or disinfectants at all times to avoid damage to health and to the parts surfaces. In particular when using flammable or igniting cleaners or disinfectants (e.g. cleaning agents containing alcohol).

5.1 Cleaning

- Small stains and dust is best removed with moist wipes.
- Larger stains can be removed by non-scouring, mild cleaning agents.

5.2 Disinfection

If disinfection is required during use, only agents according to the **Days Healthcare** skin protection and hygiene plan must be used.

6.0 Maintenance / Spare parts

6.1 Maintenance

The shower chair is maintenance-free, however, should be checked on a monthly basis.

Carry out the following tests:

- Visual check of all parts for deformation, damage, wear and tear and corrosion.
- Check all connections (nuts, screws etc.) for completeness, function and perfect fit.

The stool must not be used further if defaults are detected during use.

6.2 Spare parts

Spare parts can be obtained from your **Days Healthcare** specialist dealer. Only original **Days Healthcare** spare parts may be used (See dealer stamp on reverse of the envelope).

7.0 Reuse

The **Days Healthcare** shower chair is designed for reuse. Clean and disinfect prior to reuse. Please observe the **Days Healthcare** skin protection and hygiene plan (order no E1-05-001-000).

GB

8.0 Disposal

The shower chair consists of metal and plastic parts. Disposal of the individual materials must be carried out according to the regional environmental and disposal provisions.

9.0 Name plate

The name plate is located on the rear cross bar of the shower chair.

10.0 Specifications

max. load.....	190 kg
Weight and dimensions*:	
Total weight.....	2.7 kg
Total height (with handle and adjustable feet)	18 cm
Total width	69 cm
Total depth	32 cm

*Dimensions of the assembled shower chair, minimal differences possible.

11.0 Warranty

Days Healthcare offers a 12 months warranty on perfect quality within the framework of its general terms and conditions.

(Dealer's stamp)

better **days** *by*
atoform[®]

A T O F O R M G m b H
Zur Lauterhecke 34 • D-63877 Sailauf
Tel. 06093.944-0 • Fax 06093.944-444
w w w . a t o - f o r m . c o m

a **DCC** company