DUSCHSITZ

better **days** by
atoform[®]

D

GB

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**Duschsitz
Bonaire**

CE

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BE-07

CE

The following manual provides descriptions relating to the assembly, operation and care of the **BE-07** shower seat.

If you have any other questions about this shower stool, please contact the **Days Healthcare** dealer.

Contents

1.0	Safety instructions	4
1.1	Symbols used	4
1.2	Use in accordance with the requirements	4
1.3	Safety - General notes	5
1.4	Notes on the assembly of the shower seat	5
2.0	Wall mounting	6
2.1	Notes	6
2.2	Preparation	6
2.3	Wall mounting	8
3.0	Fold in shower seat	9
4.0	Cleaning and disinfection	10
4.1	Cleaning	10
4.2	Disinfection	10
5.0	Maintenance	10
6.0	Re-use	11
7.0	Disposal	11
8.0	Name plate	11
9.0	Specifications	11
10.0	Warranty	11
11.0	Spare parts	12

1.0 Safety instructions

1.1 Symbols used

The following symbols are used in the manual:

Caution!

This symbol identifies safety information which notifies you of hazards when dealing with the product.

NOTE

You will also find information about dealing with the product under this symbol.

1.2 Use in accordance with the requirements

The **Days Healthcare** shower seat offer a seat option to individuals with restricted standing capacity but with still sufficient sitting capacity in order to perform daily personal hygiene in the shower area.

Possible conditions or results of injury are e.g.:

- unstable standing position / incapacity to stand due to e.g. loss of strength during malignant diseases, chronic, advanced cardiac, circulatory and pulmonary diseases

The shower seat is only intended for use in connection with standard shower trays.

The secure fastening of the shower trays, for which the use of the shower seat is intended, is required.

The shower seat must only be used in shower trays in which a secure position of the stool is guaranteed. The shower seat must only be used when fully assembled.

1.3 Safety - General notes

Please read the assembly instructions for the shower seat carefully before starting the assembly!

Do not continue to use the shower seat if defects are detected during use.

Please take care that:

- the instructions are read by all individuals who will use the shower seat.
- all individuals using the shower seat have access to the instructions at any time.

Damages resulting from a failure of observing the instructions are excluded from any warranty.

Restrictions of use!

- *Patients weighing more than **130 kg** must not use the shower seat.*

Accident risk!

- *Only use the shower seat for their intended use. They are not replacements for ladders and are not exercise equipment or toys.*
- *The shower seat is not intended for use during examinations or treatments.*
- *Use the shower seat only in technically perfect condition.*
- *Use only spare parts and accessories released by **Days Healthcare**.*
- *Do not carry out structural changes which are not approved by the manufacturer.*

1.4 Notes on the assembly of the shower seat

Risk of injury due to wrong assembly!

- *Check the perfect position and firm support of all parts after the assembly.*

Risk of injury!

- *Use only tools appropriate for the work.*
- *Carry out the work only if you are familiar with using the tools employed.*

2.0 Wall mounting

2.1 Notes

The material for wall mounting is not part of the scope of delivery.

The fastening screws should be made from rustproof metal and must not have a diameter of less than 6 mm.

The selection of the wall plugs depends on the nature of the wall and must be left to the specialist.

Required tools:

- Spirit level
- Marker
- Hammer drill
- Masonry drill; \varnothing according to plug type

2.2 Preparation

The shower seat can be mounted at 5 seat heights.

The seat height is determined by the pull out adjustable feet (1) and must be set prior to mounting the seat to the wall.

It can be adjusted via the four holes (a) in the adjustable feet (1) in increments of 2.5 cm.


H1 = 45.0 cm Seat height

H2 = 47.5 cm Seat height

H3 = 50.0 cm Seat height

H4 = 52.5 cm Seat height

H5 = 55.0 cm Seat height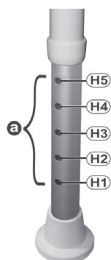

Adjusting the height:

- ➔ Remove safety clamp (2).


- ➔ Adjust stool foot to the desired setting (3) until the hole of the stool foot appears in the hole of the support tube.
- ➔ Insert safety clamp (4).



Accident risk!

- Put all stool feet into the same holes.
- Make sure that all safety clamps are tightly locked into place after the adjustment.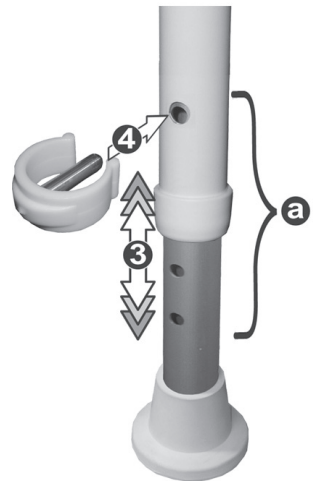


2.3 Wall mounting

- ➔ Fold stand legs and wall holder up at handle (1).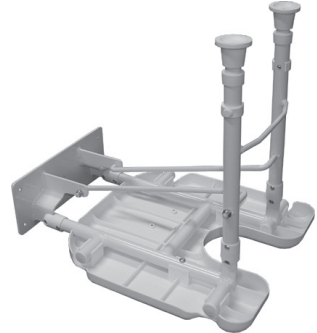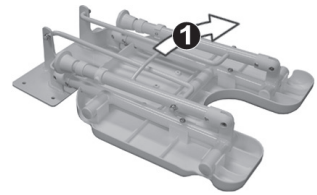

- ➔ Place shower seat with the wall bracket (1) on the wall.
- ➔ Line up the shower seat:
 - put both legs (2) firmly on the tub floor and
 - align the seat (3) horizontally (spirit level).

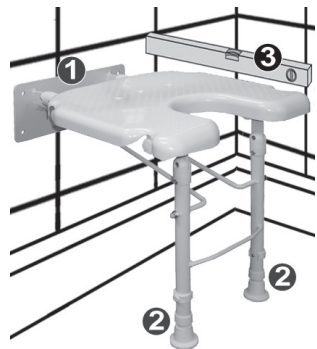
- ➔ Mark the mounting points (4) of the wall bracket and remove the shower seat.

Attention!

Check before drilling that there are no water or power lines are situated in the drill area.

- ➔ Drill plug holes, insert the selected plugs and screw shower seat tight.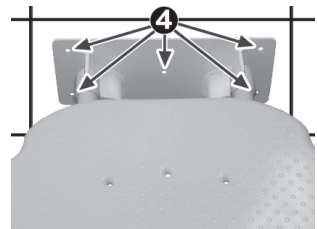

3.0 Fold in shower seat


Risk of crushing and trapping!

High risk of crushing and trapping exists for all joints of the shower seat.

The shower seat can be folded away to save space.

- ➔ To fold the shower seat up or down, hold the handle (1).

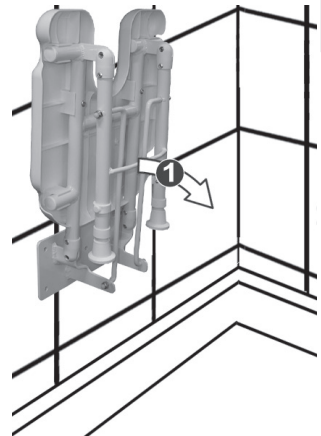GB

4.0 Cleaning and disinfection

NOTES

- Do not use tools with sharp edges (knives, metal scrapers) or aggressive solvents for cleaning.
- Observe the instructions on the cleaning agents or disinfectants at all times to avoid damage to health and to the parts surfaces. In particular when using flammable or igniting cleaners or disinfectants (e.g. cleaning agents containing alcohol).

4.1 Cleaning

- Small stains and dust is best removed with moist wipes.
- Larger stains can be removed by non-scouring, mild cleaning agents.

4.2 Disinfection

If disinfection is required during use, only agents according to the **Days Healthcare** skin protection and hygiene plan must be used.

5.0 Maintenance

The shower seat is maintenance-free, however, should be checked on a monthly basis.

Carry out the following tests:

- Visual check of all parts for deformation, damage, wear and tear and corrosion.
- Check all connections (nuts, screws etc.) for completeness, function and perfect fit.

The shower seat must not be used further if defaults are detected during use.

6.0 Re-use

The **Days Healthcare** shower seat is designed for re-use.

Clean and disinfect prior to re-use.

Please observe the **Days Healthcare** skin protection and hygiene plan (order no. E1-05-001-000).

7.0 Disposal

The shower seat consists of metal and plastic parts.

Disposal of the individual materials must be carried out according to the regional environmental and disposal provisions.

8.0 Name plate

The name plate is located on the rear cross brace of the shower seat.

9.0 Specifications

max. load 100 kg

dimensions*:

Total height 45 - 55 cm

Total width 40 cm

Total depth 50 cm

Total depth, folded 14 cm

Seat width 37 cm

Seat depth 40 cm

Number of adjustable heights 4

*Dimensions of the assembled shower seat, minimal differences possible.


10.0 Warranty

Days Healthcare offers a 24 months warranty on perfect quality within the framework of its general terms and conditions.

11.0 Spare parts

Spare parts can be obtained from your **Days Healthcare** specialist dealer. Only original **Days Healthcare** spare parts may be used (See dealer stamp on reverse of the envelope).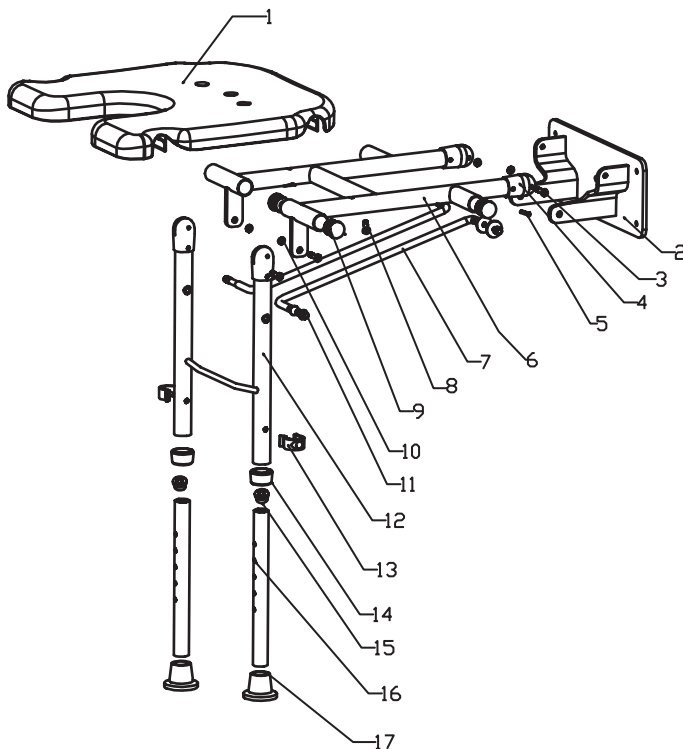

GB

Name	Item No (Qty)	Color	Order - No.
Refurbishment-kit	3 (4x) , 4 (4x), 8 (2x), 9 (6x), 10 (4x), 11 (4x), 13 (2x), 14 (2x), 15 (2x), 17 (2x)	blue white	20647 20648
E-Clip (ø 22.2mm)	13 (1x)	blue white	20659 20660
ABS - Seat square	1 (1x)	blue white	20661 20662
Leg (ø 22.2mm)	14 (1x), 15 (1x), 16 (1x), 17 (1x)	blue white	20666 20667
Tip (ø 22.2mm)	17 (1x)	blue white	20670 20671

GB

better **days** *by*
atoform[®]

A T O F O R M G m b H
Zur Lauterhecke 34 • D-63877 Sailauf
Tel. 06093.944-0 • Fax 06093.944-444
w w w . a t o - f o r m . c o m