better **days** by
atoform[®]

D

GB

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**Duschstuhl
Bonaire**

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Shower Chair
BE-03

CE

The following manual provides descriptions relating to the assembly, operation and care of the **BE-03** shower chair.

If you have any other questions about this shower stool, please contact the **Days Healthcare** dealer.

Contents

1.0	Safety instructions	4
1.1	Symbols used	4
1.2	Use in accordance with the requirements	4
1.3	Safety - General notes	5
1.4	Notes on the assembly of the shower chair	5
2.0	Component parts list	6
3.0	Assembly	6
4.0	Adjusting the chair height	9
5.0	Cleaning and disinfection	10
5.1	Cleaning	10
5.2	Disinfection	10
6.0	Maintenance	10
7.0	Re-use	11
8.0	Disposal	11
9.0	Name plate	11
10.0	Specifications	11
11.0	Warranty	11
12.0	Spare parts	12

1.0 Safety instructions

1.1 Symbols used

The following symbols are used in the manual:

GB

Caution!

This symbol identifies safety information which notifies you of hazards when dealing with the product.

NOTE

You will also find information about dealing with the product under this symbol.

1.2 Use in accordance with the requirements

The **Days Healthcare** shower chair offer a seat option to individuals with restricted standing capacity but with still sufficient sitting capacity in order to perform daily personal hygiene in the shower area.

Possible conditions or results of injury are e.g.:

- unstable standing position / incapacity to stand due to e.g. loss of strength during malignant diseases, chronic, advanced cardiac, circulatory and pulmonary diseases

The shower chair is only intended for use in connection with standard shower trays.

The secure fastening of the shower trays, for which the use of the shower chair is intended, is required.

The shower chair must only be used in shower trays in which a secure position of the chair is guaranteed. The shower chair must only be used when fully assembled.

1.3 Safety - General notes

Please read the assembly instructions for the shower chair carefully before starting the assembly!

Do not continue to use the shower chair if defects are detected during use.

Please take care that:

- the instructions are read by all individuals who will use the shower chair.
- all individuals using the shower chair have access to the instructions at any time.

Damages resulting from a failure of observing the instructions are excluded from any warranty.

Restrictions of use!

- *Patients weighing more than 130 kg must not use the shower chair.*

Accident risk!

- *Only use the shower chair for their intended use. They are not replacements for ladders and are not exercise equipment or toys.*
- *The shower chair is not intended for use during examinations or treatments.*
- *Use the shower chair only in technically perfect condition.*
- *Use only spare parts and accessories released by **Days Healthcare**.*
- *Do not carry out structural changes which are not approved by the manufacturer.*

1.4 Notes on the assembly of the shower chair

Risk of injury due to wrong assembly!

- *Check the perfect position and firm support of all parts after the assembly.*

Risk of injury!

- *Use only tools appropriate for the work.*
- *Carry out the work only if you are familiar with using the tools employed.*

GB

2.0 Component parts list

The shower chair is packed in cartons. Use the following list to check the contents of the package before starting the assembly. Please contact your medical supply shop if defects are apparent or if parts are missing.

Contents:


- 2 x (1) Side panel
- 1 x (2) Seatframe
- 1 x (3) Cross strut
- 1 x (4) Seat
- 1 x (5) Backrest
- 6 x (6) Allen screw
(M6 x 21 mm, size 5 mm)
- 2 x (7) Allen screw
(M6 x 33 mm, size 5 mm)


3.0 Assembly

Tools required:

- 1 x allen key, size 5 mm

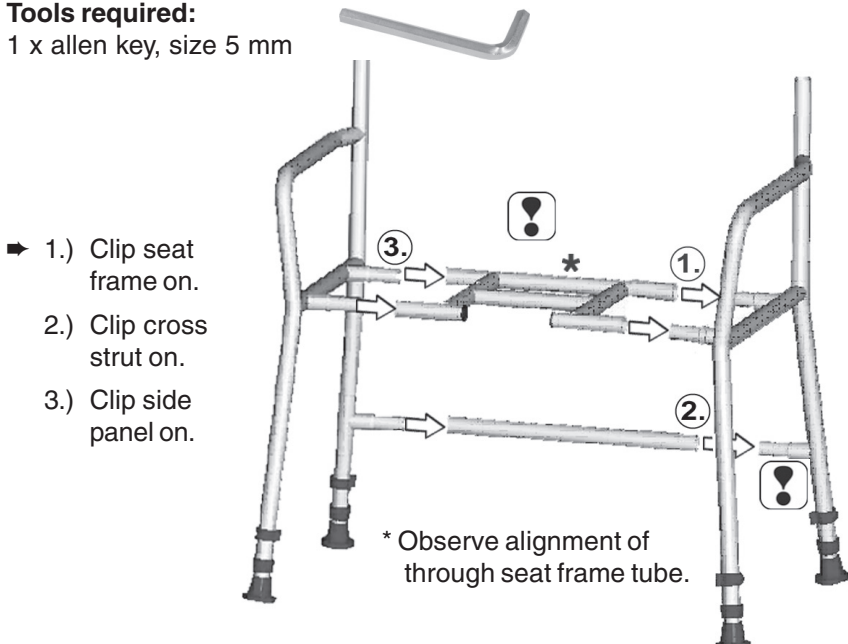
- ➔ Bolt seat frame and cross strud on (M6 x 21).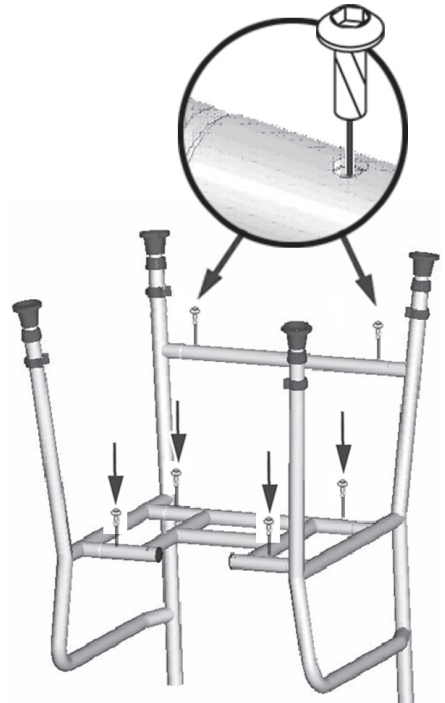


GB

- ➔ 1.) Clamp seat on and
- 2.) bolt on (M6 x 33).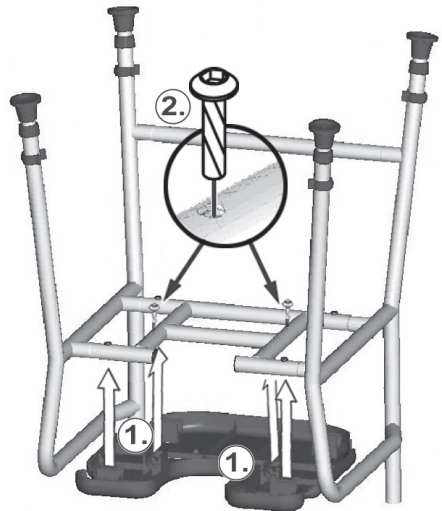

➔ Clip backrest on.

GB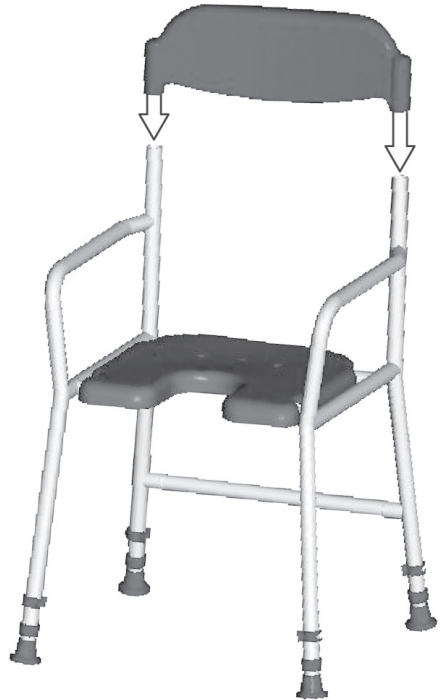

4.0 Adjusting the chair height


The chair height is adjusted with the slide-out chair feet (1).

The five holes (a) in the chair feet can be used to adjust it by 10 cm maximum.


Adjusting the height:

➔ Remove safety clamp (2).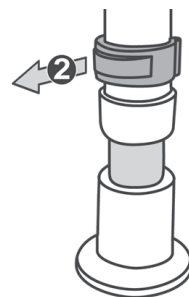

➔ Adjust chair foot to the desired setting (3) until the hole of the chair foot appears in the hole of the support tube.

➔ Insert safety clamp (4).

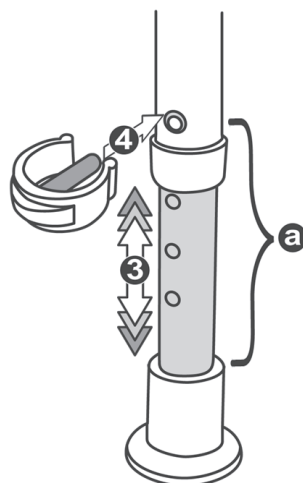Accident risk!

- Put all chair feet into the same holes.
- Make sure that all safety clamps are tightly locked into place after the adjustment.

5.0 Cleaning and disinfection

NOTES

- Do not use tools with sharp edges (knives, metal scrapers) or aggressive solvents for cleaning.
- Observe the instructions on the cleaning agents or disinfectants at all times to avoid damage to health and to the parts surfaces. In particular when using flammable or igniting cleaners or disinfectants (e.g. cleaning agents containing alcohol).

GB

5.1 Cleaning

- Small stains and dust is best removed with moist wipes.
- Larger stains can be removed by non-scouring, mild cleaning agents.

5.2 Disinfection

If disinfection is required during use, only agents according to the **Days Healthcare** skin protection and hygiene plan must be used.

6.0 Maintenance

The shower chair is maintenance-free, however, should be checked on a monthly basis.

Carry out the following tests:

- Visual check of all parts for deformation, damage, wear and tear and corrosion.
- Check all connections (nuts, screws etc.) for completeness, function and perfect fit.

The chair must not be used further if defaults are detected during use.

7.0 Re-use

The **Days Healthcare** shower chair is designed for re-use.

Clean and disinfect prior to re-use.

Please observe the **Days Healthcare** skin protection and hygiene plan (order no. E1-05-001-000).

8.0 Disposal

The shower chair consists of metal and plastic parts.

Disposal of the individual materials must be carried out according to the regional environmental and disposal provisions.

9.0 Name plate

The name plate is located on the rear cross brace of the shower chair.

10.0 Specifications

max. load 130 kg

dimensions*:

Total height 83.5 - 93.5 cm

Total width 56 cm

Total depth 56 cm

Seat height 47 - 57 cm

Seat width 40 cm

Seat depth 40 cm

Arm rest height (seat to arm rest) 20 cm

Backrest height (seat to backrest) 36 cm

Number of adjustable heights 5

*Dimensions of the assembled shower chair, minimal differences possible.

11.0 Warranty


Days Healthcare offers a 24 months warranty on perfect quality within the framework of its general terms and conditions.

12.0 Spare parts

Spare parts can be obtained from your **Days Healthcare** specialist dealer. Only original **Days Healthcare** spare parts may be used (See dealer stamp on reverse of the envelope).

GB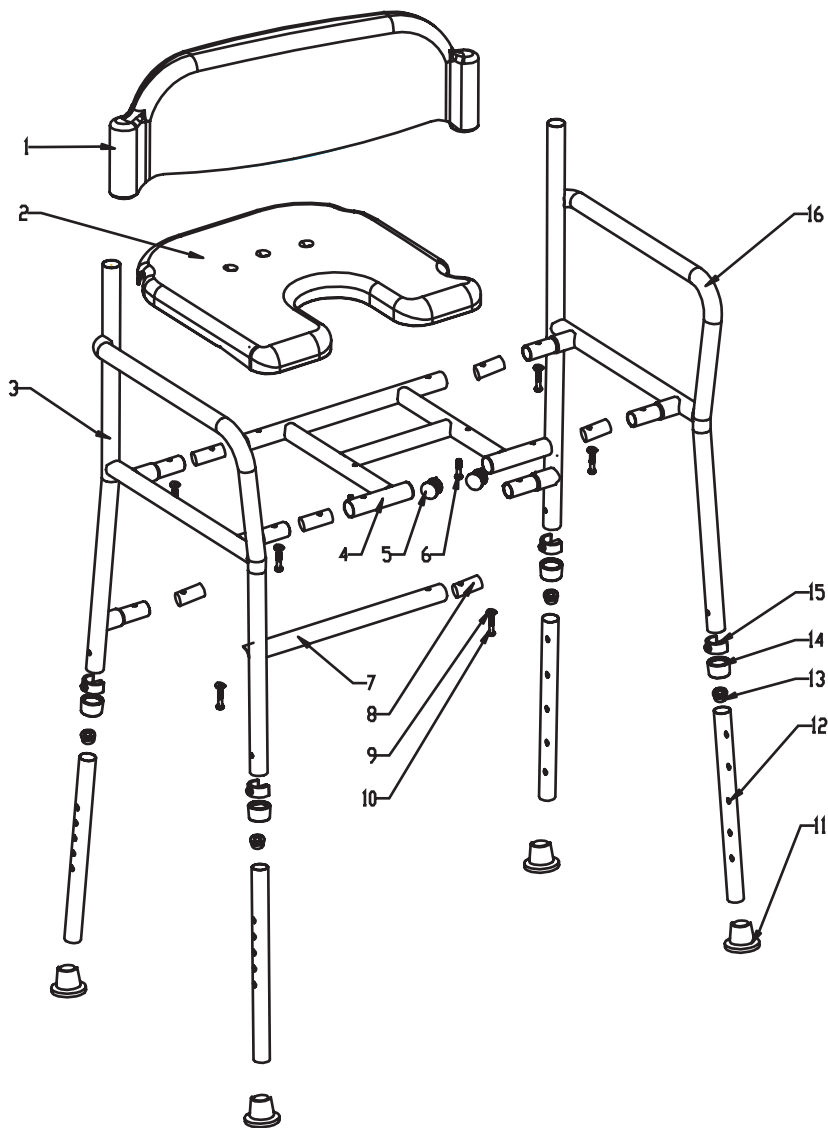

Name	Item No (Qty)	Color	Order - No.
Refurbishment-kit	5 (2x), 6 (2x), 8 (6x), 9 (6x), 10 (6x), 11 (4x), 13 (4x), 14 (4x), 15 (4x), 17 (2x), 18 (2x)	blue white	20641 20642
ABS - Backrest	1 (1x)	blue white	20655 20656
E-Clip (ø 22.2mm)	15 (1x)	blue white	20659 20660
ABS - Seat square	2 (1x)	blue white	20661 20662
Leg (ø 22.2mm)	11 (1x), 12 (1x), 13 (1x), 14 (1x)	blue white	20666 20667
Tip (ø 22.2mm)	11 (1x)	blue white	20670 20671

better **days** *by*
atoform[®]

A T O F O R M G m b H
Zur Lauterhecke 34 • D-63877 Sailauf
Tel. 06093.944-0 • Fax 06093.944-444
w w w . a t o - f o r m . c o m

a **DCC** company