**ECKDUSCHHOCKER**

better **days** by  
**atoform**<sup>®</sup>

D

GB

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**Eckduschhocker  
Bonaire**

CE

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***Corner shower stool***  
***BE-11***

**CE**

The following manual provides descriptions relating to the assembly, operation and care of the **BE-11** Corner shower stool.

If you have any other questions about this Corner shower stool, please contact the **Days Healthcare** dealer.

## Contents

|             |                                                         |           |
|-------------|---------------------------------------------------------|-----------|
| <b>1.0</b>  | <b>Safety instructions</b> .....                        | <b>4</b>  |
| 1.1         | Symbols used .....                                      | 4         |
| 1.2         | Use in accordance with the requirements .....           | 4         |
| 1.3         | Safety - General notes .....                            | 5         |
| 1.4         | Notes on the assembly of the Corner shower stools ..... | 5         |
| <b>2.0</b>  | <b>Component parts list</b> .....                       | <b>6</b>  |
| <b>3.0</b>  | <b>Assembly</b> .....                                   | <b>6</b>  |
| <b>4.0</b>  | <b>Adjusting the stool height</b> .....                 | <b>8</b>  |
| <b>5.0</b>  | <b>Cleaning and disinfection</b> .....                  | <b>9</b>  |
| 5.1         | Cleaning .....                                          | 9         |
| 5.2         | Disinfection .....                                      | 9         |
| <b>6.0</b>  | <b>Maintenance</b> .....                                | <b>9</b>  |
| <b>7.0</b>  | <b>Re-use</b> .....                                     | <b>10</b> |
| <b>8.0</b>  | <b>Disposal</b> .....                                   | <b>10</b> |
| <b>9.0</b>  | <b>Name plate</b> .....                                 | <b>10</b> |
| <b>10.0</b> | <b>Specifications</b> .....                             | <b>10</b> |
| <b>11.0</b> | <b>Warranty</b> .....                                   | <b>10</b> |
| <b>12.0</b> | <b>Spare parts</b> .....                                | <b>12</b> |

## 1.0 Safety instructions

### 1.1 Symbols used

The following symbols are used in the manual:

GB

#### Caution!

This symbol identifies safety information which notifies you of hazards when dealing with the product

#### NOTE

You will also find information about dealing with the product under this symbol.

### 1.2 Use in accordance with the requirements

The **Days Healthcare** Corner shower stool offer a seat option to individuals with restricted standing capacity but with still sufficient sitting capacity in order to perform daily personal hygiene in the shower area.

Possible conditions or results of injury are e.g.:

- unstable standing position / incapacity to stand due to e.g. loss of strength during malignant diseases, chronic, advanced cardiac, circulatory and pulmonary diseases

The Corner shower stool are only intended for use in connection with standard shower trays.

The secure fastening of the shower trays, for which the use of the Corner shower stool is intended, is required.

The Corner shower stool must only be used in shower trays in which a secure position of the stool is guaranteed.

The Corner shower stool must only be used when fully assembled.

## 1.3 Safety - General notes

**Please read the assembly instructions for the Corner shower stools carefully before starting the assembly!**

Do not continue to use the Corner shower stool if defects are detected during use.

Please take care that:

- the instructions are read by all individuals who will use the Corner shower stool.
- all individuals using the Corner shower stool have access to the instructions at any time.

Damages resulting from a failure of observing the instructions are excluded from any warranty.

### Restrictions of use!

- *Patients weighing more than **130 kg** must not use the Corner shower stools.*

### Accident risk!

- *Only use the Corner shower stools for their intended use. They are not replacements for ladders and are not exercise equipment or toys.*
- *The Corner shower stools are not intended for use during examinations or treatments.*
- *Use the Corner shower stools only in technically perfect condition.*
- *Use only spare parts and accessories released by **Days Healthcare**.*
- *Do not carry out structural changes which are not approved by the manufacturer.*

## 1.4 Notes on the assembly of the Corner shower stools

### Risk of injury due to wrong assembly!

- *Check the perfect position and firm support of all parts after the assembly.*

### Risk of injury!

- *Use only tools appropriate for the work.*
- *Carry out the work only if you are familiar with using the tools employed.*


## 2.0 Component parts list

The Corner shower stools are packed in cartons. Use the following list to check the contents of the package before starting the assembly. Please contact your medical supply shop if defects are apparent or if parts are missing.

GB

### Contents:

- 1 x (1) Seat
- 1 x (2) Seatframe
- 3 x (3) Stool leg
- 3 x (4) Allen screw  
(M6 x 21 mm, size 5 mm)
- 3 x (5) Washer
- 2 x (6) Allen screw  
(M6 x 33 mm, size 5 mm)

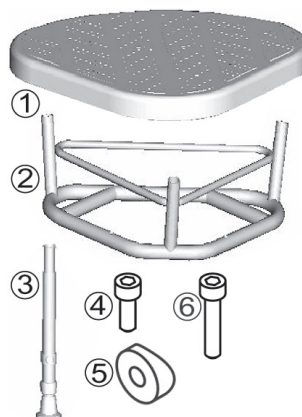
## 3.0 Assembly

### Tools required:


- 1 x allen key, size 5 mm


➔ Clip stool legs in.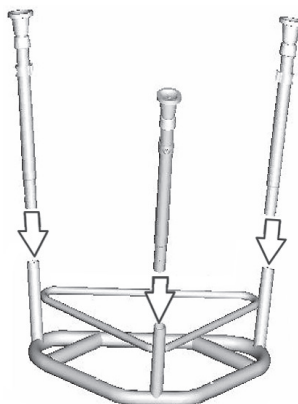

- ➔ Bolt stool legs on (M6 x 21).

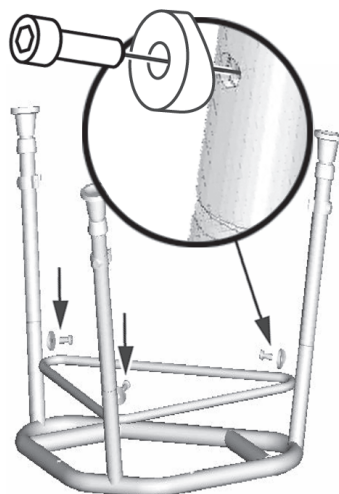- ➔ 1.) Clamp seat on  
and  
2.) bolt on (M6 x 33).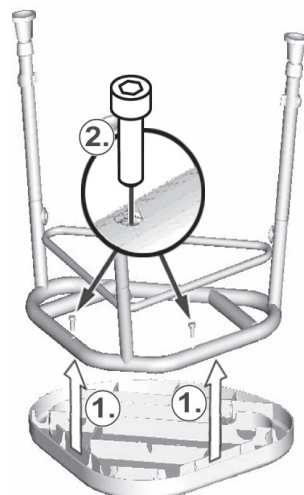

### 4.0 Adjusting the stool height


The stool height is adjusted with the slide-out stool feet (1).

The five holes (a) in the stool feet can be used to adjust it by 10 cm maximum.


#### Adjusting the height:

- ➔ Remove safety clamp (2).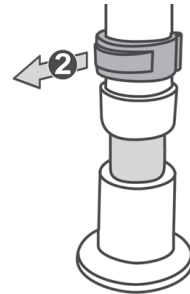

- ➔ Adjust stool foot to the desired setting (3) until the hole of the stool foot appears in the hole of the support tube.
- ➔ Insert safety clamp (4).

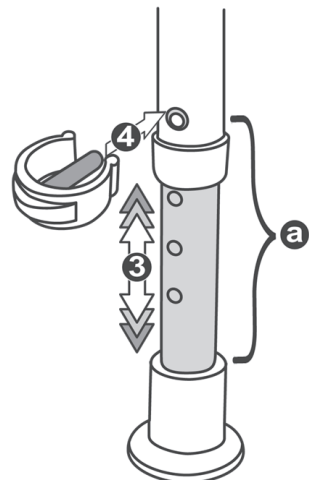#### Accident risk!

- Put all stool feet into the same holes.
- Make sure that all safety clamps are tightly locked into place after the adjustment.

## 5.0 Cleaning and disinfection

### NOTES

- Do not use tools with sharp edges (knives, metal scrapers) or aggressive solvents for cleaning.
- Observe the instructions on the cleaning agents or disinfectants at all times to avoid damage to health and to the parts surfaces. In particular when using flammable or igniting cleaners or disinfectants (e.g. cleaning agents containing alcohol).

GB

### 5.1 Cleaning

- ➔ Small stains and dust is best removed with moist wipes.
- ➔ Larger stains can be removed by non-scouring, mild cleaning agents.

### 5.2 Disinfection

If disinfection is required during use, only agents according to the **Days Healthcare** skin protection and hygiene plan must be used.

## 6.0 Maintenance

The Corner shower stools are maintenance-free, however, should be checked on a monthly basis.

### Carry out the following tests:

- ➔ Visual check of all parts for deformation, damage, wear and tear and corrosion.
- ➔ Check all connections (nuts, screws etc.) for completeness, function and perfect fit.

The stool must not be used further if defaults are detected during use.

## 7.0 Re-use

The **Days Healthcare** Corner shower stools are designed for re-use. Clean and disinfect prior to re-use. Please observe the **Days Healthcare** skin protection and hygiene plan (order no. E1-05-001-000).

GB

## 8.0 Disposal

The Corner shower stool consists of metal and plastic parts. Disposal of the individual materials must be carried out according to the regional environmental and disposal provisions.

## 9.0 Name plate

The name plate is located on the rear cross brace of the Corner shower stool.

## 10.0 Specifications

max. load ..... 130 kg

### dimensions\*:

Seat height ..... 44 - 54 cm

Seat width ..... 52 cm

Seat depth ..... 37 cm

Number of adjustable heights ..... 5

\*Dimensions of the assembled Corner shower stools, minimal differences possible.

## 11.0 Warranty

**Days Healthcare** offers a 24 months warranty on perfect quality within the framework of its general terms and conditions.

# *Corner shower stool*


---

GB

## 12.0 Spare parts

Spare parts can be obtained from your **Days Healthcare** specialist dealer. Only original **Days Healthcare** spare parts may be used (See dealer stamp on reverse of the envelope).

GB

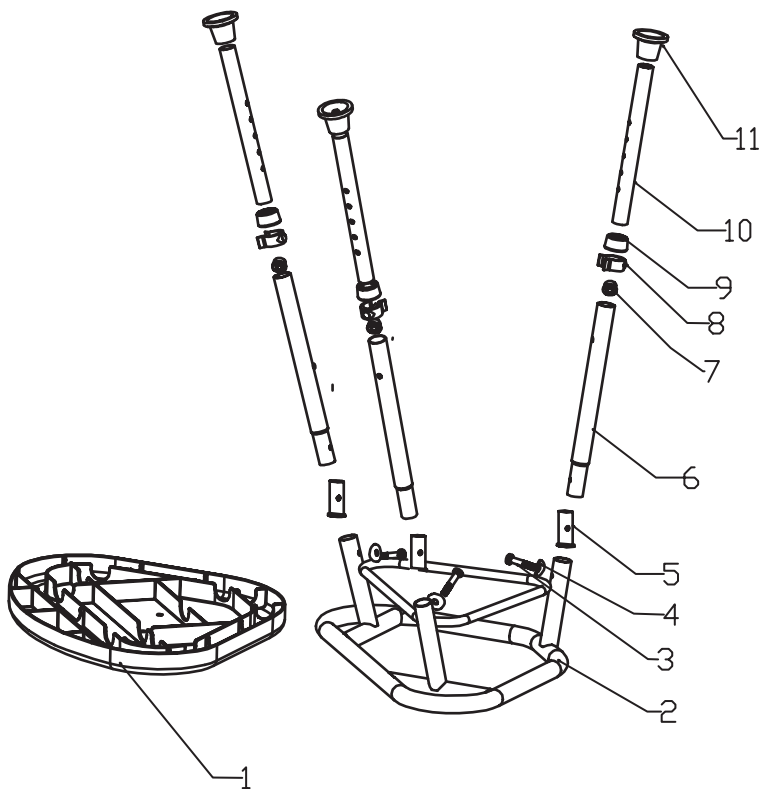| <b>Name</b>              | <b>Item No (Qty)</b>                                          | <b>Color</b>  | <b>Order - No.</b> |
|--------------------------|---------------------------------------------------------------|---------------|--------------------|
| Refurbishment-kit        | 3 (3x), 4 (3x), 5 (3x),<br>7 (3x), 8 (3x), 9 (3x),<br>11 (3x) | blue<br>white | 20653<br>20654     |
| E-Clip (ø 22.2mm)        | 8 (1x)                                                        | blue<br>white | 20659<br>20660     |
| ABS - Seat<br>triangular | 1 (1x)                                                        | blue<br>white | 20663<br>20664     |
| Leg (ø 22.2mm)           | 9 (1x), 7 (1x), 10 (1x),<br>11 (1x)                           | blue<br>white | 20666<br>20667     |
| Tip (ø 22.2mm)           | 11 (1x)                                                       | blue<br>white | 20670<br>20671     |

**GB**

*better* **days** *by*  
**atoform**<sup>®</sup>

A T O F O R M G m b H  
Zur Lauterhecke 34 • D-63877 Sailauf  
Tel. 06093.944-0 • Fax 06093.944-444  
w w w . a t o - f o r m . c o m